


Quick Start Guide

Luna Eye



V1.0



1. Luna Eye Overview

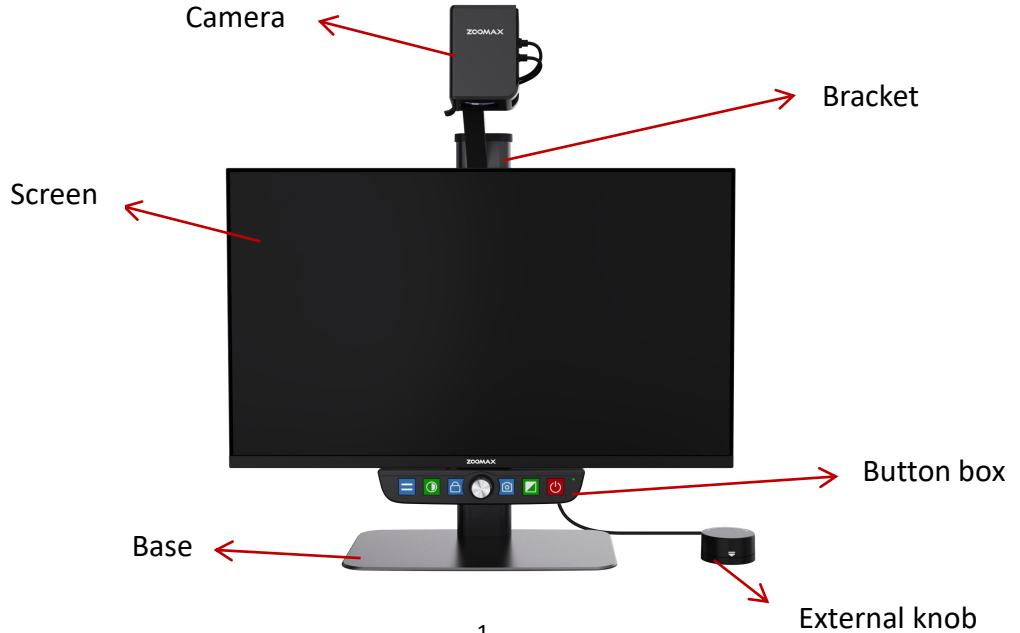

2.Start the Luna Eye

- ◆ Connect Power Supply: Connect the power supply with the matching adapter.
- ◆ Connect External Knob: Connect the external knob to port on the button box interface with USB-C cable.
- ◆ Power On: Long press the power button  on the button box for 2 seconds until you hear the buzzer.
- ◆ Power Off: Long press the power button  on the button box for 2 seconds to power off.

3.Basic Functions and Ports on the Button Box

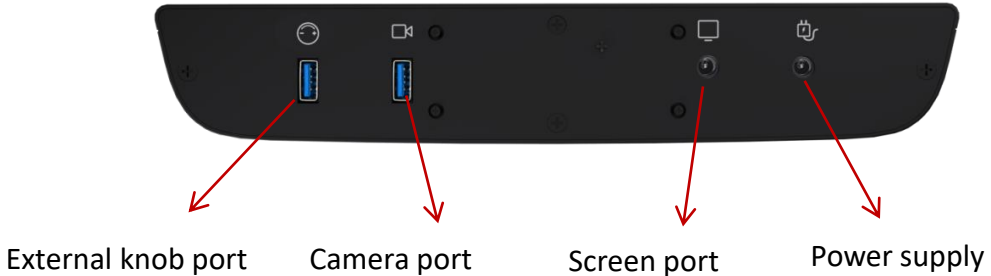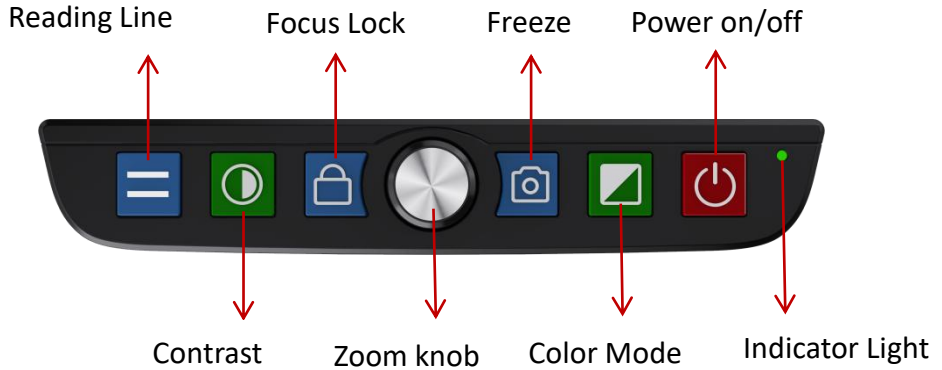

4. Use the External knob



- ◆ Zoom in: Rotate clockwise
- ◆ Zoom out: Rotate counterclockwise
- ◆ Freeze: Short press down

5. Other functions

Frequency	Long press the zoom knob button call out and short press to change
Color mode setting	Short press the contrast button and color mode button and rotate the zoom knob to choose
Reading line position	Hold the reading line button while rotating the zoom knob
Panning	Pan left and right: Hold the freeze button and rotate the zoom knob
	Pan up and down: Hold the focus lock button and rotate the zoom knob
Gallery	Double-tap the freeze button to open/exit the gallery
Delete the image	Short press the zoom knob when view the image
Restore	Long press the freeze button for 10s

ZOOMAX TECHNOLOGY CO., LIMITED

Email: support@zoomax.com

Visit us at: www.zoomax.com