



Quick Guide

SNOW 12

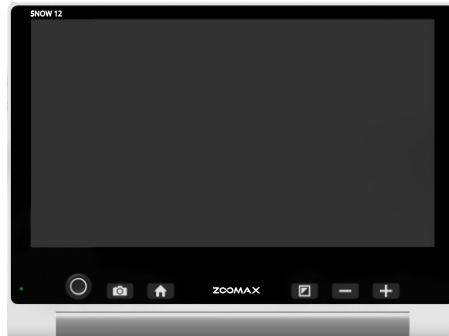


V1.2

► Buttons

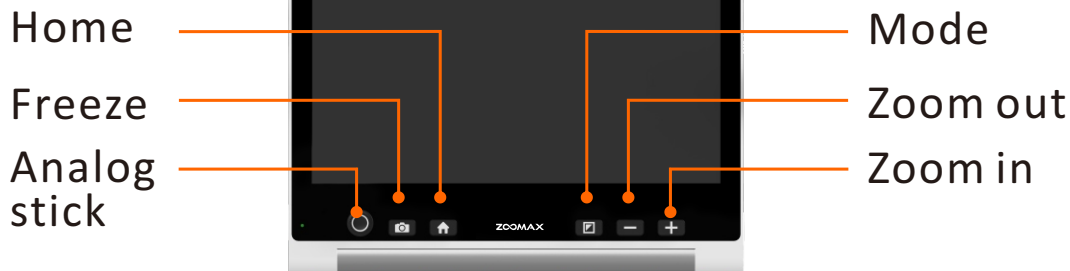


➤ Buttons

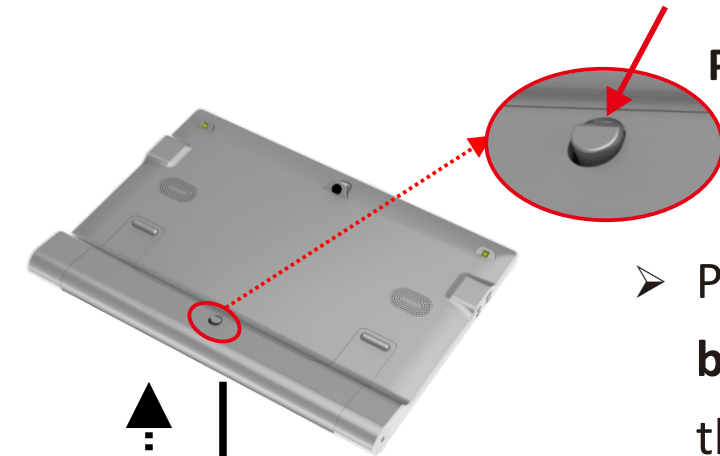


Push button

- Press the **Push button** to open the Stan Ffoot.

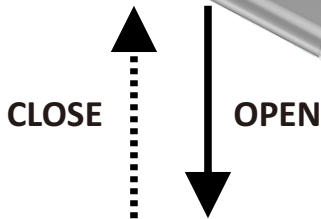


Stand Foot

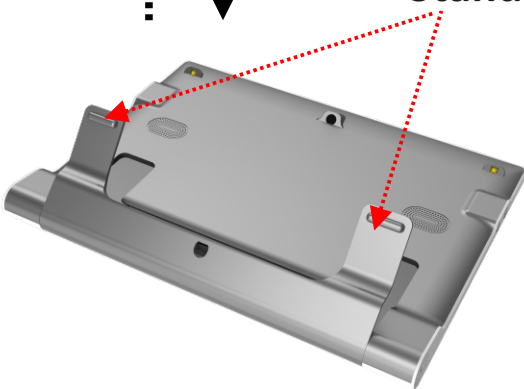


- Push the **Stand Foot** back to close the stand.


➤ Foldable Stand

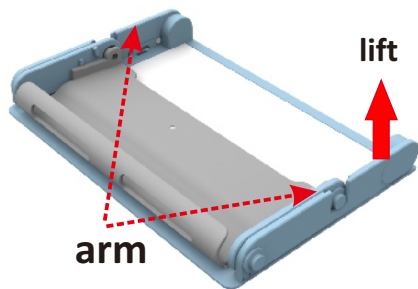

Step 1

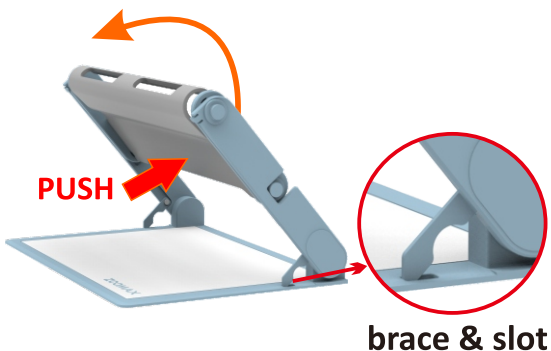

Step 2

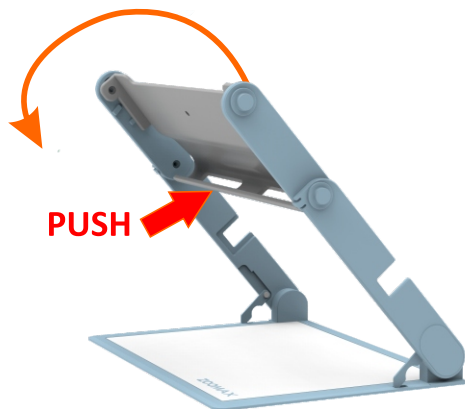

Step 3



Step 4

➤ Snow 12 with Foldable Stand

Close the Stand Foot and put the Snow 12 on the Foldable Stand.



①



②

① Snow 12 can be used alone.

② Snow 12 can be also used with “Foldable Stand”.

Basic Function

Operation



Turn on/off the device

Press power button



Zoom in/out

Press or



Open the stand

Press the “Push button” on the bottom of the back cover



Change color mode

Short press

Change to full color fast

Press and hold for 1 second



Freeze image

Short press



Move image with analog stick

Move the analog stick up/down/right/left to move the image magnified



Reading line or masks

Press and simultaneously in real-time viewing, or choose in setting

Advanced Function Operation

	Main menu	Short press the home button  to enter/exit main menu.
		Move the analog stick up/down/right/left to cycle through the icons. Press  to confirm. Press  to go back.
		Or tap the icons in main menu
	Save photo	Tap  in the bottom icon bar after freezing the image.
	Check saved photo	Tap the gallery icon  in main menu.
	Setting	Tap the setting icon  in main menu.
	System information	Tap the system information icon  in main menu.
	Adjust the intensity of contrast	Press and hold  first and short press  or  , or choose in setting.

Text-to-speech Function Operation



OCR & Speech

Press and hold the  for 3 seconds to enter/exit OCR & Speech function.

Or tap the top left corner icon  in main menu to enter it.

Place the document flat under the unit .
Press  to capture the image and process the document to speech.
Press  to go back.

ZOOMAX TECHNOLOGY CO., LIMITED

Add.: 9F, Building D, Paradise Software Park,
No.3 Xidoumen Road, Xihu District, Hangzhou, China

Tel: +86-571-87006308

Fax: +86-571-87397220

Email: sales@zoomax.com

Visit us at: www.zoomax.com